Sistema Cervical Pediátrico HALO No Invasivo Modelo C-24 Lerman

Descripción:

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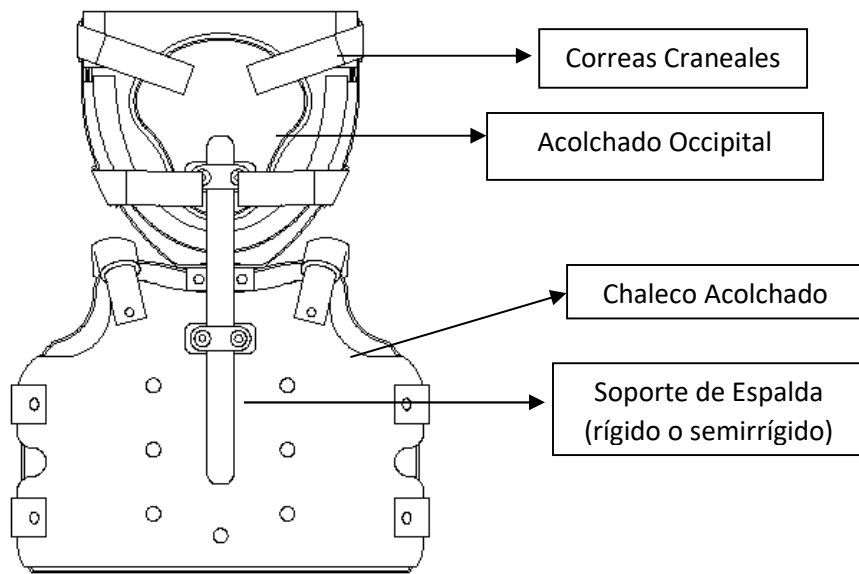
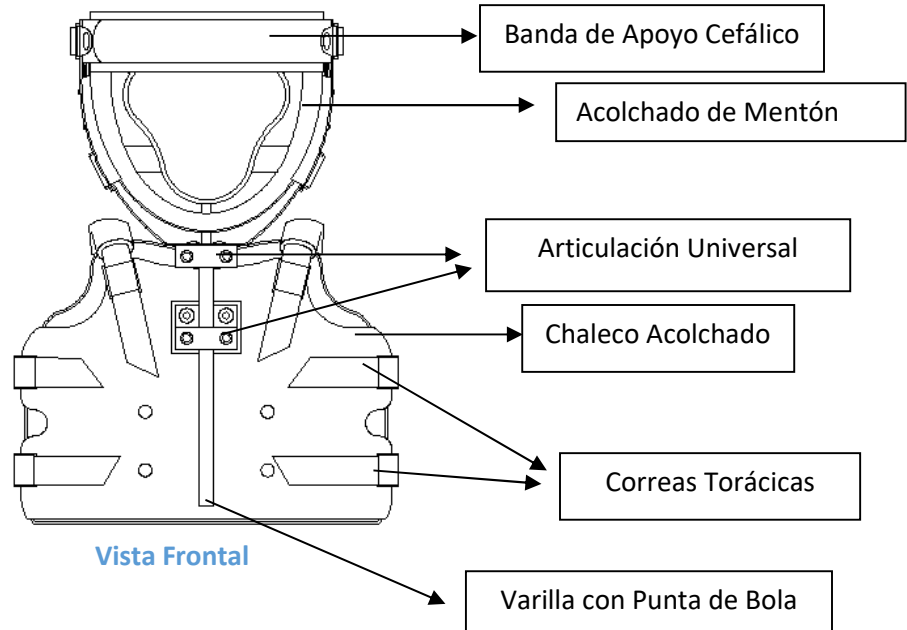
Fitting Instructions for Lerman Noninvasive Pediatric Halo System

CONTRAINDICATIONS: Should not be used when immobilization of a structurally unstable cervical spine is required.

1. Remove the anterior and posterior head supports.
2. Fit the thoracic section, utilizing standard TLSO fitting techniques. Check for any signs of redness or skin impingement after application.
3. Fit the facial portion. The forehead band can be adjusted for height. The vertical support must be anterior to the ear. The width of the facial horseshoe bracket can be adjusted by hand.
4. Fasten the occipital portion to the facial portion using the two straps on each side. Straps may be trimmed to length after fitting.
5. Connect the anterior support bar. Adjust for height and head position.
 - a. For more involved torticollis cases, an angled anterior support is provided
6. Connect the posterior support bar. Tighten all locking screws.
 - a. Use the plastic bar as a semi-rigid support, e.g. torticollis.
 - b. Use the aluminum bar as a rigid support when more restriction of movement is required.

Liners may be hand washed in cold water. Air dry.

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12/10/2020

EC REP

Acorn Regulatory Consultancy
Services Limited
Knockmorris Cahir Co. Tipperary
Ireland, Postcode: E21 R766
P 353 52 617 6706
F 353 52 614 8806

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BECKER ORTHOPEDIC
A Tradition of Excellence A Commitment to Innovation

EUROPEAN DECLARATION OF CONFORMITY

Competent Authority: Health Products Regulatory Authority
Kevin O'Malley House
Earlsfort House,
Earlsfort Terrace,
Dublin 2
Ireland
Telephone: 353 1 676 4971
Fax: 353 1 676 7836

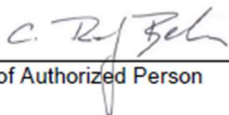
Manufacturer: Becker Orthopedic Appliance Company US-MF-000004002
635 Executive Drive
Troy, Michigan 48083 U.S.A.
Telephone: (248) 588-7480
Fax: (248) 588-2960

Product: C-24 Lerman Noninvasive Pediatric HALO System
Class: I (non-sterile, non-measuring)

Basic UDI-DI: 194545C24SP

Location of European Design Dossier: Acorn Regulatory Consultancy Services Limited IE-AR-000000592
Knockmorris
Cahir
Co. Tipperary
Ireland
Postcode: E21 R766
Telephone: 353 52 617 6706
Fax: 353 52 614 8806

The undersigned hereby declares, on behalf of Becker Orthopedic Appliance Company of Troy, Michigan (USA), that the above-referenced product(s), to which this declaration relates, is in conformity with the provisions of: EU Regulation 2017/745 Annex IV of May 26, 2017 concerning Medical Devices. Conformity assessment per Recital 60 of 2017/745. This declaration is issued under the sole responsibility of the manufacturer.

Signature of Authorized Person

VICE PRESIDENT
Title

C. RUDOLF BECKER
Name (Typed or Printed)

7/21/2023
Date
Expires 5 years from date signed

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Becker Orthopedic
635 Executive Drive
Troy, Michigan 48083-4576
p 800-521-2192 | 248-588-7480
f 800-923-2537 | 248-588-2960
BeckerOrthopedic.com   